



## Instructions for Use (IFU) Animal Health & Welfare Pathway - Blood Sample Kit

### Your kit should contain:

- 5x red top vacutainers with QR code labels attached (plus 1 spare vacutainer)
- 5x 20G x 1" needles
- 1x vacutainer holder
- 1x biohazard bag
- 1x white absorbent pad
- 1x UN3373 box
- 1x security seal for the box
- 1x mailing bag with seal
- 1x freepost label (*if postage included kit option selected on ordering*)
- 1x test required label (on inside lid of box)

### Complete a Cattle Animal Health & Welfare Pathway Review & Follow Up with VetDX

- 1) Collect 5 blood samples in each of the vacutainer tubes
- 2) Place samples in biohazard bag → UN3373 box
- 3) Enter on VetDX

**IMPORTANT: Complete VetDX before it arrives at the laboratory to ensure timely analysis - VetDX is the submission form for the lab**

  - a) Log in at [vetdxs.app](https://vetdxs.app) → Select **Visits** → Click **Schedule a Visit**
  - b) Select **farm to be visited** + **Cattle Pathway ("Lab Name") – Animal Health & Welfare Review** visit type + complete the information
  - c) Click **Schedule a Visit** → **Start Visit**
  - d) Click **Record Samples** → enter or scan QR codes for each blood sample → **Add** → **End Visit**
- 4) Seal UN3373 box with security sticker → place in mailing bag → post in a Royal Mail box/office (or if using own/lab provided courier arrange collection as per your standard process).  
Post the same day if possible. If not, refrigerate 1–2 days  
Do not post on Friday
- 5) Results appear in VetDX once analysis has been completed by the laboratory - you will receive a notification email once results are available (once available make any edits then **Publish** the report)

### Contact/support

Questions about collection, VetDX, or results → [support@vetdxs.com](mailto:support@vetdxs.com)